



## Instructions for audiologists

1. Our Inearz custom in ear monitors are full shell in ear monitors. What we need from the audiologist is a full shell impression. A full shell impression includes the full helix, crus of the helix, tragus and anti-tragus.
2. We require that the audiologist use a high viscosity silicone impression material.
3. If at all possible please make sure that they use cotton ear dams instead of foam. The foam can stick to the material and remove critical parts of the ear canal.
4. Make sure that the ear dam is pushed into the ear canal past the second bend. We also would like an open mouth impression. This can be accomplished with a 1 inch bite block, or by simply just moving your mouth (as if you were singing) while the impression material is hardening.

### **FISHER HEARING TECHNOLOGIES INC.**

105 Commerce Street  
Suite 125  
Lake Mary, FL 32746

Phone: 407-585-0554

Fax: 407-585-0555

E-mail: [Kim@fisherhearing.com](mailto:Kim@fisherhearing.com)

[www.inearz.com](http://www.inearz.com)

Please feel free to contact us with any questions,  
or comments.